



AcceleDent™ Trial Mouthpiece Set  
SKU NUMBER 105000  
TRAY STERILIZATION INSTRUCTIONS

A sterilization tray with Trial Mouthpieces is provided in the AcceleDent Starter Kit.



Select the size of Trial Mouthpiece that best fits the patient's upper arch in terms of width and anterior-posterior dimension. Optimize occlusal contact by evaluating the three profile options for the selected size.

The Trial Mouthpiece must be sterilized prior to use as follows:

- Steam Sterilization with pre-vacuum: 3 - 4 minutes at 270°F (132°C) and 28-30 psi (193 – 207 kpa), 30 minute dry time.\*
- Steam Sterilization with gravity displacement: 15 minutes at 270°F (132°C) and 15 -17 psi (104 – 117 kpa), 30 minute dry time.\*

The sterilization instructions have been validated to a Sterility Assurance Level (SAL) of  $10^{-6}$  in accordance with USFDA (21 CFR Part 58) Regulations and ISO 17664 and ISO 17665-1 using the biologics indicator overkill method.

\*Please follow the manufacturer's sterilization wrap Directions For Use



OrthoAccel Technologies, Inc.  
Office: 866.435.0111  
Email: sales@orthoaccel.com

[www.AcceleDent.com](http://www.AcceleDent.com)